



Infection Prevention with Confidence!



In the war against microbes, Accel® Disinfectants are the first line of defense for Infection Prevention. Accel® provides comfort and confidence that commonly touched surfaces and non-critical instruments are rendered safe to use.

Accel TB is a Ready-To-Use One-Step Disinfectant Cleaner for use as a daily cleaner and disinfectant of environmental surfaces and patient care devices. Accel TB is a powerful disinfectant that uses the Accelerated Hydrogen Peroxide® (AHP®) technology and is effective against a broad-spectrum of pathogens in only 5 minutes!



EFFECTIVE AGAINST NOROVIRUS

EFFICACY OF ACCEL TB

TUBERCULOCIDAL (5 min.): *Mycobacterium terrae*

GENERAL VIRUCIDE (5 min.): Proven effectiveness against the Polio Virus, Sabin strain type 1 which allows for a General Virucide claim against most enveloped and non-enveloped viruses, Influenza A H1N1 (This product has demonstrated effectiveness against Poliovirus and is expected to inactivate all influenza A viruses including 2009 [H1N1] pandemic Influenza A virus), Human Immunodeficiency Virus (HIV), Norovirus (Feline Calicivirus as surrogate)

BACTERICIDAL (5 min.): *Pseudomonas aeruginosa*, *Staphylococcus aureus*, *Salmonella choleraesuis*

FUNGICIDAL (5 min.): *Trichophyton mentagrophytes*

BROAD-SPECTRUM SANITIZING (30 sec.): *Pseudomonas aeruginosa* , *Staphylococcus aureus* , *Salmonella choleraesuis* , *Staphylococcus aureus* (Methicillin resistant, MRSA), *Enterococcus faecalis* (Vancomycin resistant, VRE), *Acinetobacter baumannii*

DIRECTIONS FOR USE (HEAVILY SOILED SURFACES REQUIRE CLEANING PRIOR TO DISINFECTION)

DISINFECTION OF NON-CRITICAL, NON-POROUS HAND CONTACT SURFACES (5 Minutes)

Apply disinfectant to surface with cloth or disposable wipe. Ensure surface remains wet for 5 minutes. Wipe dry. No rinsing is required on non food contact surfaces, however, as with most chemistry, periodic damp wiping of soft surfaces and high risk equipment is considered best practice.

BROAD-SPECTRUM SANITIZING (30 Seconds)

Apply disinfectant to surface with cloth or disposable wipe. Ensure surface remains wet for 30 seconds. Wipe dry. No rinsing is required on non food contact surfaces, however, as with most chemistry, periodic damp wiping of soft surfaces and high risk equipment is considered best practice.

	Accel TB Liquid	Accel TB Wipes
Product Number	ACCDISR1-TB (1 Litre) ACCDISR4-TB (4 Litre) ACCDISR200-TB (200 Litre) - Special Order	ACCWIP1-TB (6"x7" Wipe) ACCWIP-TB10x10 (10"x10" Wipe) ACCWIP-TB-SP (Soft Pack 7.5"x10" Wipe) ACCWIP-TB-B (Slim Canister 7"x8" Wipe)
DIN #	02246903	02283352
Concentration of Active	0.5% Hydrogen Peroxide	0.5% Hydrogen Peroxide
Container Size	1L, 4L, 200L	160 sheets per canister (6"x 7") 60 sheets per canister (10" x 10") 10 sheets per Soft Pack (7.5"x10") 60 sheets per Slim Canister (7"x8")
Dilution	Ready To Use	Ready To Use
Shelf Life	24 Months	24 Months



Engineering Revolutionary Disinfectants for the War Against Microbes

1 (800) 387-7578
www.virox.com